



ENTHERMICS Medical Systems

EC230ℓ

Fluid Warming Cabinet



Specifications:

The single-chambered warming cabinet is constructed with 20 gauge stainless steel exterior casing and door. Window door allows observation of inventory while the door is closed. The cabinet is warmed using the patented Halo Heat- low-heat-density electrothermal cable array. The cabinet is equipped with one (1) white, epoxy-coated wire basket, with a 12 liter capacity to accommodate most commonly used surgical fluids packaged in bags or bottles. The cabinet is furnished with four (4) 1-1/4" (31mm) non-skid rubber feet.

Special Information:

- Right-hand door swing is standard.
- Left-hand door is available as a special order.
- Stacking combinations listed under Options and Accessories.

Control:


The chamber is controlled by one (1) electronic control which consists of a 4 digit L.E.D. display, ON/OFF key, heat indicator, INCREASE and DECREASE keys, integrated LOCK feature and a series of prompt sequence indicators. Individual temperature controls for irrigation fluids (IRR) and injection fluids (INJ) allow for precise temperature settings and can be easily set to operate in Celsius or Fahrenheit. The IRR temperature may be adjusted from 90° to 150°F (32° to 66°C), and the INJ temperature can be adjusted from 90° to 110°F (32° to 43°C). An alarm will sound if temperature exceeds 10°F (6°C) over the set-point temperature. In the event of a power failure, the cabinet will remember its programming and begin to operate as before. A warming shut-off system, separate from the electronic control, prevents overheating.

Capacity and Weights

| | |
|--------------------|----------------|
| Interior Capacity: | 12 liters |
| Net Weight: | 82 lb (37 kg) |
| Ship Weight: | 112 lb (51 kg) |

Electrical


120 V.A.C. 50/60 Hz, 1ph
650 Watts, 5.5 Amps

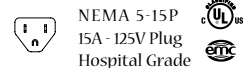

NEMA 5-15P
15A - 125V Plug
Hospital Grade



230 V.A.C. 50/60 Hz, 1ph
800 Watts, 3.5 Amps

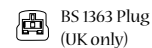

BS 1363 Plug
(UK only)



CEE 7/7
220-230V Plug



Options and Accessories

| | |
|--|---------|
| Cart with Casters – Height 34-1/2" (876mm) | 722 |
| Casters*, 3" (76mm) | 14227 |
| Casters*, 5" (127mm) | 4007 |
| Leg Kit* , 6" (152mm) | 5205 |
| Lock for Door Handle | LK-2763 |
| Stacking Hardware | CONTACT |

FACTORY

| | |
|--------------------|-------------------|
| EC230ℓ over EC230ℓ | |
| EC230ℓ over EC340ℓ | EC230ℓ over EC340 |
| EC230ℓ over EC770ℓ | EC230ℓ over EC770 |

*Platform (E5089) must be ordered as a base when ordering casters or legs for this unit.

