



# ENTHERMICS Medical Systems

## EC340l

### Fluid Warming Cabinet



#### Specifications:

The single-chambered warming cabinet is constructed with 20 gauge stainless steel exterior casing and door. Window door allows observation of inventory while the door is closed. The cabinet is warmed using the patented Halo Heat® low-heat-density electrothermal cable array. The cabinet is equipped with two (2) white, epoxy-coated wire baskets on glides, each with a 10 liter capacity to accommodate most commonly used surgical fluids packaged in bags or bottles. The cabinet is furnished with a full-perimeter rubber bumper assembly and one (1) set of 5" (127mm) heavy duty casters — two with locking brake.

#### Special Information:

- Right-hand door swing is standard.
- Left-hand door is available as a special order.
- When ordering a combination of units to stack:
  - 1) Indicate top and bottom placement.
  - 2) Order one set of stacking hardware.
  - 3) Delete full perimeter bumper (44108) and casters (CS-24874 and CS-24875) from upper unit.

#### Control:

The chamber is controlled by one (1) electronic control which consists of a 4 digit L.E.D. display, ON/OFF key, heat indicator, INCREASE and DECREASE keys, integrated LOCK feature and a series of prompt sequence indicators. Individual temperature controls for irrigation fluids (IRR) and injection fluids (INJ) allow for precise temperature settings and can be easily set to operate in Celsius or Fahrenheit. The IRR temperature may be adjusted from 90° to 150°F (32° to 66°C), and the INJ temperature can be adjusted from 90° to 110°F (32° to 43°C). An alarm will sound if temperature exceeds 10°F (6°C) over the set-point temperature. In the event of a power failure, the cabinet will remember its programming and begin to operate as before. A warming shut-off system, separate from the electronic control, prevents overheating.

#### Capacity and Weights

Interior Capacity:	20 liters
Net Weight:	171 lb (78 kg)
Ship Weight:	232 lb (105 kg)

#### Electrical

120 V.A.C. 50/60 Hz, 1ph  
775 Watts, 6.5 Amps



NEMA 5-15P  
15A - 125V Plug  
Hospital Grade



230 V.A.C. 50/60 Hz, 1ph  
700 Watts, 3.0 Amps



BS 1363 Plug  
(UK only)



CEE 7/7  
220-230V Plug



#### Options and Accessories

Cart with Casters - Height 34-1/2" (876mm)	722
Combination Lock	5004663
Cylinder Lock for Door Handle	LK-22567
Leg Kit, 6" (152mm)	44093
Stacking Hardware	CONTACT FACTORY
EC230 over EC340l	EC340l over EC340l
EC230l over EC340l	EC340l over EC770
EC340 over EC340l	EC340l over EC770l

