



# ENTHERMICS Medical Systems

## EC230

### Blanket Warming Cabinet



SHOWN WITH OPTIONAL CART WITH CASTERS

#### Specifications:

The single-chambered warming cabinet is constructed with 20 gauge stainless steel exterior casing and doors. A window door allows observation of inventory with the door closed. The cabinet is warmed using the patented Halo Heat® low-heat-density electrothermal cable array, mounted against the walls, eliminating the need for a heat circulating fan. The cabinet is equipped with one (1) white, epoxy-coated blanket support assembly. The cabinet is furnished with four (4) 1-1/4" (31mm) non-skid rubber feet.

#### Special Information:

- Right-hand door swing is standard.  
Left-hand door is available as a special order.
- Stacking combinations listed under Options and Accessories.

#### Control:

The chamber is controlled by one (1) electronic control which consists of a 4 digit L.E.D. display, ON/OFF key, heat indicator, L.E.D., increase and decrease keys, built-in lock-out feature and a series of prompt sequence indicators. The electronic control can easily be set to operate in Fahrenheit or Celsius and has an adjustable temperature range of 90° to 200°F (32° to 94°C). Timer key allows the user to program the control to automatically turn on and turn off once during a 24 hour period at selected times. In the event of a power failure, the cabinet will remember its programming and begin to operate as before. A warming shut-off system, separate from the electronic control, is an additional safety feature.

#### Capacity and Weights

Interior Capacity:	2.3 cubic feet (c. 0.07m <sup>3</sup> )
Net Weight:	76 lb (34 kg)
Ship Weight:	106 lb (48 kg)

#### Electrical

120 V.A.C. 50/60 Hz, 1ph  
600 Watts, 5.5 Amps

NEMA 5-15P  
15A-125V Plug  
Hospital Grade



230 V.A.C. 50/60 Hz, 1ph  
750 Watts, 3.5 Amps

BS 1363 Plug  
(UK only)

CEE 7/7  
220-230V Plug



#### Options and Accessories

Cart with casters - Height 34-1/2" (876mm)	722
Casters*, 3" (76mm)	4227
Casters*, 5" (127mm)	4007
Leg Kit*, 6" (152mm)	5205
Stacking Hardware	CONTACT FACTORY
EC230 over EC230ℓ	EC230 over EC230
EC230 over EC340ℓ	EC230 over EC340
EC230 over EC770ℓ	EC230 over EC770

\*Platform (E5089) must be ordered as a base when ordering casters or legs for this unit.

