



OSOM

TEST FOR PROFESSIONAL USE



Mono Test

*A simple, in-office test for the detection of mononucleosis.
Easy to use with fast, accurate results within minutes.*

✓ **EASY-TO-USE**

Fits easily into any practice or lab.

✓ **FAST**

Results in 5 minutes.

✓ **VERY ACCURATE**

>99% sensitivity; 96% specificity.

✓ **EASY-TO-READ**

Two-color results: blue test line, red control line.

✓ **CONVENIENT**

Compact packaging with everything you need to run 25 tests. Kit includes positive and negative controls. Room temperature storage.



TEST PROCEDURE / RESULTS

For serum, plasma or whole blood samples in collection tubes: use the transfer pipette provided and add 1 drop to the test tube.

For fingertip blood: after filling a capillary tube end-to-end, dispense all the blood into the test tube.



Add 1 drop of diluent to the bottom of the test tube. Mix.



Place the test stick into the treated sample. Leave the test stick in the test tube.



Read results at 5 minutes. Positive results may be read as soon as red control line appears.

