



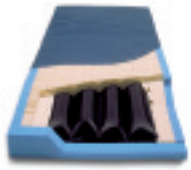






SURFACES

	<b>Comfort and Prevention</b> • Low to moderate risk Braden 13-26	<b>Prevention and Low Level Treatment</b> • High risk (Braden $\leq 12$ ), or • Multiple Stage I's to Single Stage II on trunk or pelvis	<b>Aggressive Treatment or Prevention</b> • High risk (Braden $\leq 12$ ), or • Multiple Stage II's or existence of any deeper ulcers	<b>Critical Treatment or Prevention</b> • High risk (Braden $\leq 12$ ), or • Multiple, more severe ulcers	
<b>PressureGuard Turn Select®</b> E0277 			Can't/won't reposition frequently <b>and</b> has $\leq 2$ ulcer-free weight bearing positions <b>or</b> can't be positioned off of ulcer(s)	Manual positioning prohibited by pain <b>or</b> can't be positioned off of ulcer(s) <b>and</b> requires customized turn pattern or >10 minute cycles	Powered Treatment
<b>PressureGuard Easy Air®</b> E0277 			Excessive skin moisture (maceration) exacerbates negative effects of pressure <b>or</b> Can't tolerate repositioning/rotation	Excessive skin moisture (maceration) exacerbates negative effects of pressure <b>or</b> Can't tolerate repositioning/rotation	
<b>PressureGuard® APM/APM<sup>2</sup></b> E0277 			Can't/won't reposition frequently <b>and</b> has $\leq 2$ ulcer-free weight bearing positions <b>and</b> breakdown does not respond to lower level products	Can be repositioned manually <b>and</b> can be positioned off of ulcer(s)	
<b>PressureGuard CFT®</b> E0373 		Can't/won't reposition frequently <b>and</b> can be positioned off of ulcer(s)	Repositions frequently or infrequently <b>and</b> has $\geq 2$ ulcer-free weight bearing positions <b>and</b> can be positioned off of ulcer(s)	Can be repositioned manually <b>and</b> can be positioned off of ulcer(s)	Non-Powered Treatment
<b>PressureGuard Renew®</b> 		Can't/won't reposition frequently <b>and</b> can be positioned off of ulcer(s)			
<b>Geo-Mattress®</b> E0184 	Physical profile indicates need for protection	Reposition frequently by self or caregiver <b>and</b> can be positioned off of ulcer			Non-Powered Prevention
<b>Geo-Matt® Overlay</b> 	Physical profile indicates need for protection	Reposition frequently by self or caregiver <b>and</b> can be positioned off of ulcer			